

GE

# Voluson E6 BT17

## Data Sheet

This datasheet is valid with Software Version BT17 Ext. 4 and higher

### Product description

The Voluson\* E6 BT17 is an advanced imaging platform that combines extraordinary image quality with our superb volume ultrasound technology.

### Highlights

- High Resolution LCD LED Display 23" inches
- Based on the Radiance System Architecture
- Advanced 4D
- HD*live*\*
- Advanced VCI
- SonoBiometry
- SonoAVC\**follicle*
- SonoVCAD\**heart*
- STIC
- Anatomical M-Mode
- Wide Sector
- Coded Contrast Imaging†
- Scan Assistant
- SonoNT\*
- SonoIT
- Elastography
- Elastography Analysis
- Elastography Ratio Measurement
- B-Flow\*
- SonoRender*live*
- Electronic Height Adjustment
- Floating User Interface
- On Board Archive including Preview and Pre-selection



## General Specifications

### Dimensions and weight

Height (minimum)	1330 mm (52.4 in)
Height (maximum)	1670 mm (60.0 in)
Adjustable	with electrical motor
Width	580 mm (22.8 in)
Depth	940 mm (37.0 in)
Weight (no Peripherals)	147 kg (324.1 lbs.)

### Power supply

Voltage	100 – 240 VAC
Frequency	50/60 Hz (+/-1%)
Power	Max. 800 VA Including all options typical power consumption ~350VA without peripherals
Thermal Output	max. 2730 BTU/h

### Console design

4 Active and Imaging Probe Ports	
Integrated HDD	500 GB
Operating System: Microsoft Windows 7 Embedded	
Integrated DVD+R(W)/CD-R(W) drive	
On-board storage for Peripherals	
Wheels	Wheel diameter 150 mm
Integrated cable management	
Front and rear handles	
Probe port illumination	

## User Interface

### Operator keyboard

Floating Keyboard:	
Rotation: adjustable +/- 38° from center	
Height adjustable + 195 mm (7.9 in)	
Full-sized, backlit alphanumeric keyboard	
Ergonomic hard key layout	
Interactive back-lighting	
Integrated recording keys for remote control of up to 6 Peripherals or DICOM® devices, one dedicated DVD recording key	

### Touch screen

12.1 in High Resolution color LCD screen	
Multi touch interactive dynamic software menu	
Brightness adjustable	

### Monitor

23 in High Res LCD LED Display with DVI interface	
Resolution Full HD 1920 x 1080 pixel	
High brightness with 300 cd/m <sup>2</sup>	
Tilt/Rotate Adjustable Monitor	
Tilt angle: min. +30°/-75°	
Hor. rotate Angle: +/- 90°	
Digital brightness and contrast adjustment. Five default settings available: Extra Dark, Dark-, Semi Dark-, Light-, Extra Light Room, each for a higher and lower color temperature	

## System Overview

### Exam types

Abdominal
Obstetrical
Gynecological
Small Parts and Breast
Vascular
Pediatrics
Transrectal
Cardiology
Cephalic
Musculoskeletal (MSK)

### Operating modes

Brightness Mode (B-Mode) (2D)
Motion Mode – M-Mode (conventional M-Mode)
Anatomical M-Mode (AMM)
Pulsed wave Doppler (PW) with HPRF
Continuous Wave Doppler imaging (CW)
Color Flow Doppler mode (CFM)
Power Doppler Mode (PD)
High Definition Power Doppler (HD-Flow*)
Tissue Doppler mode (TD)
B-Flow (BF)
Elastography
Contrast Imaging Mode†
Combination modes: M/CF, M/HD-Flow, M/TD
Extended View (XTD View)
Volume Mode (3D/4D):
• 3D Static
• 4D Real Time
• VCI-A
• VCI-OmniView
• Spatio-Temporal Image Correlation (STIC)
• 4D Biopsy

### Scanning methods

Electronic Sector
Electronic Convex
Electronic Linear
Mechanic Volume Sweep

### User Authentication and Logging Functionality

Multiple Users with individual log on credentials
Different and adjustable access levels
LDAP Interface
Audit Trail and Usage Log

### Privacy and Security Functionality

Harddisc AES Encryption with 256 bit length
Automatic Logoff
Encrypted DICOM Communication Capability (TLS)
Encryption and Data Anonymization Export Capability
All ports, services and shared resources that are not required for the intended use are disabled
Operating System Access disabled

## System Overview (cont.)

### Transducer types

Sector Array
Convex Array
Microconvex Array
Linear Array
Active Matrix linear Array (1.5D)
Volume probes 4D:
• Convex Array
• Microconvex Array
• Linear Array

### System standard features

Innovative user interface with high resolution 12.1 in LCD touch panel
B-Mode
M-Mode
PW-Doppler
CFM (Color Flow Doppler Mode)
Automatic Tissue Optimization
Auto TGC
Coded Harmonic Imaging with Pulse Inversion Technology
Coded Excitation (CE)
HD-Flow & Power Doppler Mode
B-Flow
Tissue Doppler
XTD
SRI II (Speckle Reduction Imaging)
HD <i>live</i>
CrossXBeam <sup>CRI*</sup> (Compound Resolution Imaging)
SonoNT
SonoIT
SonoBiometry
SonoRender <i>live</i>
Scan Assistant:
• Includes measurements, annotations and fetal anatomy worksheet entries
• Performs predefined mode changes
Standardize image sequence upon DICOM transfer
DICOM 3.0 Connectivity
Static 3D Mode:
• B Mode only
• B + Power Doppler Mode
• B + CFM Doppler Mode
• B + HD-Flow Mode
• B + CRI
• B + CRI + CFM
• B + CRI + PD
• B + CRI + HD-Flow
• B + B-Flow
Focus and Frequency Composite (FFC)
High Resolution Zoom (HD Zoom)
Pan Zoom
Steering
Virtual Convex
Wide Sector
Beta-View
Patient information database
Image Archive on hard drive
3D/4D data compression (lossy/lossless)
Inversion Mode
Real-time automatic Doppler calculations

Measurement and Calculations including Worksheets/Report for:
• OB
• GYN
• Vascular
• Cardio
• Abdominal
• Small Parts
• Transrectal
• Pediatrics
• Cephalic
• Musculoskeletal (MSK)
Multigestational Calculations
Integrated uplink for Cloud-based data storage (Tricefy™) (not available in all countries)

### System options

Advanced 4D
VOCAL II
Advanced VCI (Volume Contrast Imaging)
SonoVCAD <i>labor</i>
SonoVCAD <i>heart</i>
Elastography
Basic STIC
SonoAVC, including SonoAVC follicle, antral and general CW Mode, including steerable CW on C1-5D and C4-8D
Anatomical M-Mode (AMM)
Coded Contrast Imaging + 3D HyCoSy†
Integrated Software DVR
• Digital recording
• One drive for data export and recording
• DVD Formats: DVD+R, -R, +RW, -RW for recording, DVD and CD support for data export
• USB support: FAT32 compatibility
GYN IOTA LR2 Model. May not be available in all countries

### Peripheral options

Integrated printers:
• B&W thermal printer
• Color thermal printer
External Color desktop printer with network printing capabilities & connection kits
ECG Digital Module
Foot Switch, with programmable functionality
Barcode Scanner

## System Overview (cont.)

### Display modes

Simultaneous capability in combination with SRI and/or CRI:

- B+PW
- B+CFM, B+PD, B+TD
- B+HD-Flow
- B+M, B+AMM
- B+3D, B+4D
- B+CRI
- B+SRI
- B+CRI+SRI
- Contrast<sup>†</sup>+SRI
- B+CRI/3D+CRI
- B+SRI/3D+SRI
- B+CRI+SRI/3D+CRI+SRI
- B+CRI/4D+CRI
- B+SRI/4D+SRI
- B+CRI/STIC+CRI
- B+SRI/STIC+SRI
- B+CRI+SRI/STIC+CRI+SRI
- B/B+CRI
- B/B+SRI
- B/B+SRI+CRI
- B/CFM+CRI
- B/CFM+SRI
- B/CFM+CRI+SRI
- B/PD+CRI
- B/PD+SRI
- B/PD+CRI+SRI
- B/HD-Flow+CRI
- B/HD-Flow+SRI
- B/HD-Flow+CRI+SRI

Real-time Triplex Mode:

- B/CFM/PW
- B/PD/PW
- B/HD-Flow/PW

Selectable alternating Modes:

- B+PW or CW
- B/CFM+PW or CW
- B/PD+PW or CW
- B/HD-Flow+PW or CW
- B+CFM or PD or HD-Flow or CW

Multi-image (split, quad):

- Live and/or frozen
- Split: B+B, B/CFM+B/CFM or B/PD+B/PD or B/TD+B/TD or B/HD-Flow + B/HD-Flow or BF+BF, Contrast<sup>†</sup>+Contrast<sup>†</sup>
- Split simultan: B+B/CFM or B+B/PD or B+B/HD-Flow
- Split: B+PW or M or CW
- Split: Frame Review/XTD-View
- Quad: B+B+B+B or BF or Contrast<sup>†</sup>, B/CFM+B/CFM+B/CFM+B/CFM or B/PD or B/TD or B/HD-Flow
- Independent Cine playback
- Quad: A+B+C+3D or 4D
- TUI: 1x1, 1x2, 2x2, 3x2, 3x3, 3x4, 4x4
- Segmentation: quad (A/B/C/Segm. Object), single (Segm. Object)
- Split: TUI Overview+1 slice
- Zoom Read/Write (with or without overview image)

• Image Size:

- Standard format
- XL format

Colorized Image:

- Colorized B
- Colorized M
- Colorized PW
- Colorized 3D

Time line display:

- Independent Dual B/PW Display
- Display Formats: Top/Bottom selectable format (Size: 1/2:1/2; 1/3:2/3; 2/3:1/3)

### Display annotation

Patient Name:

- Last: max 62 characters
- First: max 62 characters
- Middle: max 62 characters

ID: max 32 characters

Secondary patient ID (Citizen Service Number)

Accession #: max 16 characters

Hospital Name: max 30 Characters

Sonographer

Gestational age (OB) or LMP (GYN)

Birth date

Date: 3 Types selectable:

- MM/DD/YYYY
- DD/MM/YYYY
- YYYY/MM/DD

Time display selectable: 12/24 hours

Probe Name

Application Name

Gray Scale bar

Depth Scale

Focal Zone Marker

Frame Rate

Zoom Start/Depth

B-Mode:

- User Preset
- Receiver Frequency
- Gain
- Dynamic Control
- Gray Map
- Edge Enhance
- Persistence
- SRI, CRI
- Probe Orientation

M-Mode/AMM –Mode:

- Gain
- Dynamic control
- Edge Enhance
- Reject
- M-Cursor, AMM-Cursor
- Time Scale

Doppler Mode:

- Acoustic Power
- Gain
- Angle
- Sample Volume Depth and Width
- Wall Motion Filter
- Doppler Frequency
- Velocity or Frequency Scale
- Spectrum Inversion
- Time Scale
- PRF
- HPRF

Color Flow Imaging modes (CFM, PD, TD, HD-Flow):

- Acoustic Power
- Color Gain
- Color Balance
- Color Balance Marker
- Quality
- Wall Motion Filter
- PRF
- Color Map
- Color Scale: kHz, cm/s, m/s
- Power and Symmetrical Velocity Imaging
- Color Velocity Range
- Spectrum Inversion

## System Overview (cont.)

### Display annotation (cont.)

3D/4D Mode:

- 3D/4D Sub Program
- Threshold
- Quality
- Volume Box Angle
- Mix
- Acquisition Mode
- Compression
- Orientation Markers
- TUI: slice distance (0.5-10mm)
- TUI: slice position in overview image
- SonoVCAD<sup>heart</sup>
- STIC acquisition time
- Calculated heart rate for STIC

Elastography Mode:

- Acoustic Output
- Tx Frequency
- Transparency
- Velocity Range
- Elasto Map
- Persistence
- Line Density

TGC Curve

Cine Frame Number

Recorder Status

Body Pattern: 117 types organized in 10 anatomical groups

Measurement Results

Displayed Acoustic Output:

- TIS: Thermal Index Soft Tissue
- TIC: Thermal Index Cranial (Bone)
- TIB: Thermal Index Bone
- MI: Mechanical Index

Predefined Biopsy Guide Line

ECG Line

Trackball function (Trackball and Trackball buttons)

GE Logo

Zoom overview image (zoom box position)

## System parameters

### System setup

User Programmable Preset Capability, User program etc.

Languages: English, French, German, Spanish, Italian, Danish, Dutch, Finnish, Norwegian, Swedish, Russian, Japanese, Simplified Chinese

EUM Languages: English, German, Spanish, Italian, French, Russian

Free programmable Scan assistant lists including Add, Delete, Edit and Reorder of checklist items

Up to 800 Programmable Annotations organized in 10 anatomical groups

6 programmable Px buttons for documentation preferences like Save, DICOM Send, Print, Check, Cine length, jpeg, etc.

Several user configurable functions:

- Clinic Name
- Display (TGC curve, Screen Lock, Screensaver, Auto Scan Stop, Beeper, 3D/4D Screen Controls)
- Trackball speed
- Zoom Overview window
- Title bar settings
- Start Exam and End Exam configuration
- Dim function
- Patient Info display

### Measure setup

M&A Setup including Add, Delete, Edit and Reorder of measure items

Application Setup including several parameters of Measurement, Doppler Trace and Calculation presets

Global Setup including several parameters of Measurement, Cursor and Result window presets

### Biopsy setup

User programmable needle guidelines

### Pre-processing

Write Zoom up to 8x

B/M-Mode:

- Gain
- TGC
- Dynamic Range
- Acoustic Output
- Transmission Focus Position
- Transmission Focus Number
- Transmission Frequency
- Persistence Control
- Line Density Control
- Reject
- Sweep Speed
- M-Cursor position

PW-Mode:

- Gain
- Dynamic Range
- Acoustic Output
- Transmission Frequency
- PRF
- Wall Motion Filter
- Sample Volume Gate
- Length, Depth, Pos
- Velocity Scale
- Sweep Speed

Color Flow Imaging Modes (CFM, PD, TD, HD-Flow)

- Gain
- Acoustic Output
- PRF
- Wall Motion Filter
- Line density
- Ensemble
- Dynamic
- Smooth (Rise and Fall)
- Frequency
- Balance
- Line Filter
- Quality
- Artifact Suppression

### Post-processing

Read Zoom: 0.8x – 3.4x Zoom (with HD-Zoom functionality up to 22x Zoom)

B-Mode:

- 2D Gain
- Dynamic Contrast
- Gray Map
- Edge Enhance
- Colorized B
- SRI II (Speckle Reduction Imaging)

M-Mode:

- Gray Map
- Colorized M
- Edge Enhance
- Display Format
- Sweep Speed

PW Mode:

- Gray Map
- Baseline Shift
- Angle Correction
- Colorized D
- Scale (kHz, m/s, cm/s)
- Trace
- Invert
- Sweep Speed

Color Flow Imaging Modes (CFM, PD, TD, HD-Flow):

- Display Threshold
- Display Mode (V,V-T,T,P,P-T) (CFM only)
- Color Map
- Scale (CFM and HD-Flow)
- Baseline

## System Overview (cont.)

### Post-processing (cont.)

B-Flow

- Gray map
- Colorized B-Flow
- Advanced SRI (Speckle Reduction Imaging)
- Dynamic Contrast

### Image processing and presentation

Digital Beamformer

2.192.763 system processing channel technology

Minimum Depth of Field: 0 – 1 cm (Zoom, probe dependent)

Maximum Depth of Field: 0 – 36 cm (probe dependent)

Transmission Focus: 1-5 Focus Points selectable (probe and application dependent)

Focal Zone position, up to 10 positions selectable

Continuous Dynamic Receive Focus/ Continuous Dynamic Receive Aperture

256 gray levels

16.8 million Colors 24 bit

Up to 265 dB Dynamic Range

Image reverse: Right/Left

Rotation: 0°, 180°

### Cine features

Cine Features:

- Dual/Quad image CINE Display
- CINE Gauge and CINE image number display
- CINE Review Loop
- Selectable CINE Sequence for CINE Review (by Start Frame and End Frame)
- Side Change in dual CINE Mode
- Measurements /Calculations & Annotations on CINE

Length:

- 2D: 512MB: up to 10 min (depending on B-image size and FPS); typical: about 3 min/4000 images (with curved array: 15cm depth, angle 81°, 22 FPS)
- M-Mode: 32MB: up to 1 min motion time (depending on sweep and depth)
- PW/CW-Mode: 32MB: up to 1 min motion time (depending on sweep speed)

Cine operation:

- Manual: image by image
- Auto run: speed: 25 to 200% of real-time rate, play repeat mode: forward-forward, forward-backward-forward

### Image/volume storage (archive)

Standard and fully anonymized archive available

Images stored as:

- Raw Data file (proprietary format)
- DICOM file (Single-or Multi-Frame)

Volume file stored as:

- Raw Data file (proprietary format)
- DICOM file
- Size: typically: 0.8 – 5MB (depending on probe and adjusted volume size)

Compression:

- 2D: JPEG, Lossless, high, mid low
- 3D/4D: Lossy and lossless compression available. Typical compression rates are 50% with lossless compression, 15% with lossy compression but maximum quality and 5% with lossy compression and reduced quality (approximate values).

Review of current Exam and archived data sets (Single Images and Cine Clips). View format: Raw data, DICOM data. Display Formats: 1x1, 2x2, 3x3

Reload of current/ archived data sets: 2D Raw Data (incl. Color Doppler, Spectral Doppler and M-mode). 3D Raw Data (single Volume incl. Calc. Cines). 4D Raw Data (Volume Cine).

Export as:

- Bitmap files: BMP, TIFF, JPEG;
- Raw files: RAW (2D), VOL (Volume data), 4DV (RAW, VOL incl. Patient data – password protected)
- Video File Format: AVI, MP4
- DICOM Files: DCM, DICOM Files with DICOMDIR
- 3D Raw Date: export Cartesian format possible
- Surface Formats: STL, OBJ, PLY, 3MF, XYZ

AVI Codec: MS Video 1, FullFrames

Export to: DVD+R(W), CD-R(W), Network, USB devices, email

Export Anonymous function: available for following image types: AVI, BMP, TIFF, JPEG, MP4

Backup function to: DVD+R(W)/CD-R(W), Network, USB devices

Repro function: Settings recall (e.g. Geometry, Gain, Colormap, etc.) from a stored or reloaded picture

Exam History: direct access to images from previous exams; direct access to Measure Reports images from previous exams; Image compare window on screen to compare images from previous exams with current exam image

Hard Drive Data Storage space: approx. 450 GB

### Connectivity

Ethernet network connection

SIM card modem. May not be available in all countries.

6 USB ports for USB devices

DICOM support:

- Verify
- Print
- Store
- Modality Worklist
- Structured Reporting
- Storage Commitment
- MPPS (Modality performed procedure step)
- Media Exchange
- Off network / mobile storage queue
- Query/Retrieve
- TLS

## Scanning Parameters

### B-Mode

B Acoustic Power	1-100
Scan Angle	Probe dependent
Gain range	+15 to -20 dB
Gray scale values	8 bit
SRI	5 steps (1-5)
CRI	8 steps (1-8)
CRI filter	4 steps: off, low, mid, high
CE	On/Off (Probe dependent)
FFC	On/Off (Probe dependent)
Persistence filter	8 steps (pre)
Line filter	3 steps (pre) off, low (12.5/75/12.5%), high (25/50/25%)
Line Density	3 steps (pre) low, norm, high
Reject	51 steps (pre) from 0 to 255
Enhance	6 steps 0, 1, 2, 3, 4, 5
Gray maps	21 (18 basic maps and 3 User-defined maps)
Tint maps	10
Dynamic	12 different dynamic curves C1 - C12
Display Modes	B, XTD
Max. B-Mode Frame Rate	> 3000 frames/sec
Screen Formats:	
	<ul style="list-style-type: none"> <li>• 2D Imaging: Single (B), Dual (B+B), Quad (B+B+B+B)</li> <li>• XTD View: Single (XTD), Dual (B+XTD)</li> </ul>

### M-Mode

Working Modes	M (conventional M- Mode) AMM (Anatomical M-Mode)
Power control range	1-100
Gain range	+15 to -20 dB
M sweep speeds:	
	<ul style="list-style-type: none"> <li>• 900/450/300/225/150/100 pixels/sec;</li> <li>• 26.44/13.22/8.81/6.61/4.40/2.94 cm/s in relation to system monitor</li> </ul>
Review (memory times)	>60 s (32MB)
Signal processing M:	
	<ul style="list-style-type: none"> <li>• Dynamic range: 1 to 12</li> <li>• Reject: 0 to 255</li> <li>• Enhance: 0 to 5</li> <li>• Gray maps: 18</li> <li>• Tint maps: 10</li> </ul>
Display Modes:	
	<ul style="list-style-type: none"> <li>• M: 2D+M, 2D+M/CFM, 2D+M/HD-Flow, 2D+M/PD, 2D+M/TD</li> <li>• AMM: 2D+AMM, 2D/CFM+AMM/CFM, 2D/HD-Flow+AMM/HD-Flow, 2D/TD+AMM/TD</li> </ul>
Screen Formats: (window arrangement)	
	<ul style="list-style-type: none"> <li>• 2D+M and 2D+AMM: up/down (horizontal): three different sub formats 30/70, 50/50, 70/30% left/right (vertical): 50/50%</li> <li>• 2D+AMM+AMM: left/rt-up/rt-down: 50/25/25%</li> </ul>

### M-Color Flow Mode

Acoustic MCFM Power	1-100
MCFM Color Maps	8 maps
CFM Gain	+/-15 dB range, 0,1 dB steps
CFM Velocity Scale Range	PRF: 150Hz to 20,5kHz
Wall Motion Filter	8 - 3000 Hz
Ensemble (color shots per line)	8-16, step size 1
Gentle color filter	
Smooth filter:	Rise: 12 steps Fall: 12 steps
CFM Spectrum Inversion	
CFM Baseline Shift	17 steps
Pre-settable and independently adjustable B-, M and MCFM Gain	
CFM Threshold	1 - 255 steps
Balance	25 - 225, step size 5
Artifact suppression	On/Off
Color Display Mode:	
	<ul style="list-style-type: none"> <li>• V (Velocity)</li> <li>• V-T (Velocity + Turbulence)</li> <li>• V-P (Velocity + Power)</li> <li>• T (Turbulence)</li> <li>• P-T (Power + Turbulence)</li> </ul>
Real-time Triplex Mode	B + M + MCFM in any depth

### Spectral Doppler Mode (PW, CW)

Operating Modes	PW (Pulsed Wave Doppler, Single Gate), Steerable CW (Continuous Wave Doppler)
Transmit Frequencies	PW-Doppler: 1.75..18 MHz CW-Doppler: 1.75..16 MHz
Pulse Repetition Frequency (PRF)	PW-Doppler: 0.9..22 kHz CW-Doppler: 1.3..40.0 kHz
Sample Volume (Doppler Gate)	Length: 0.7,1,2,3,4,5,6, 7,8,9,10,15 mm Position: 5 mm to B-scan end, Angle correction: -85°...0°...+85°
Power control range	1-100
Gain range	+15 to -25 dB (PW) +15 to -15 dB (CW)
WMF (wall motion filter)	PW: 30...500 Hz, CW: 30..1000 Hz
Baseline shift	± PRF/2, ± 8 steps
Spectrum Analyzer	FFT (Fast Fourier Transformation), max. 256 channels, 256 amplitude levels

## Scanning Parameters (cont.)

### Spectral Doppler Mode (PW, CW) (cont.)

PW sweep speeds	Simplex (26.44/13.22/8.81/6.61/4.40/2.94 cm/s), Duplex/ Triplex (26.44/13.22/8.81/6.61/4.40/2.94 cm/s)
Review (memory times)	>60 s(32MB)
Measurable flow velocities:	
<ul style="list-style-type: none"> <li>• PW: 1cm/s – 8m/s (<math>\alpha=0^\circ</math>, 2.0MHz, max. Baseline shift) 1cm/s – 16m/s (<math>\alpha=60^\circ</math>, 2.0MHz, max. Baseline shift)</li> <li>• CW: 1cm/s – 11.6m/s (<math>\alpha=0^\circ</math>, 2.0MHz, max. Baseline shift) 1cm/s-23.20m/s (<math>\alpha=60^\circ</math>, 2.0MHz, max. Baseline shift)</li> </ul>	
Signal processing: Dynamic range: 15 steps (10 to 40), Gray maps: 18 basic curves and 3 User-defined (pre, post) Tint maps: 11	
Scale display	Vert.: kHz, cm/s, m/s (selectable), Hor.: 1s marker (big), ½ s marker (small)
Screen Formats	2D/D: up/down (horizontal): three different sub formats 30/70, 50/50, 70/30% left/right (vertical): 50/50%
Display Formats	2D/D (duplex update, simultaneous); 2D+CFM/D, 2D+HD-Flow/D, 2D+PD/D, 2D+TD/D (triplex update, CW or PW). 2D+CFM/PW, 2D+PD/PW, 2D+HDFlow/PW, 2D+TD/PW, (triplex simultaneous, PW only)
Audio Modes	Stereo (both directions separately in both channels)
Audio Volume	Adjustable, control digipots

### Color Doppler Mode

Screen Formats	2D+CFM (Single, Dual, Quad)
Display Modes:	
<ul style="list-style-type: none"> <li>• Simultaneous dual mode: 2D/2D+CFM</li> <li>• Triplex mode: 2D+CFM/PW, 2D/M+MCFM</li> <li>• Volume Mode: 3D+CFM</li> </ul>	
Color coding:	
<ul style="list-style-type: none"> <li>• Steps: 65536 color steps</li> <li>• Display modes: V-T (velocity + turbulence), V (velocity), V-P (velocity + power), T (turbulence), P-T (power + turbulence)</li> </ul>	
Depth range	Axial: 0 to B scan range Lateral: 0 to B scan range
Baseline shift	17 steps (independent from spectral Doppler)
Inversion of color direction	Yes
Wall Motion Filter:	7 steps (low1, low2, mid1, mid2, high1, high2, max)
Smoothing Filter	12 steps rising time, 12 steps falling time

Gain Control	+15 dB to -15 dB, 0.2 dB steps
Line Density (color line density)	10 steps
Ensemble (color shots per line)	CFM: 7 to 31; MCFM: 8 to 16
Flow Resolution	4 steps (low, mid1, mid2, high)
Pulse repetition frequency	CFM: 150 Hz to 20.5 kHz MCFM: 150 Hz to 20.5 kHz
Color Map	Up to 8 different color codes for each probe
Frequency range	1 to 18 MHz depending on the probe, adjustable in 3 steps (low, mid, high)
Balance	From 25 to 225
Max. meas. velocity	4.23 m/sec
Min. meas. velocity	0.3 cm/sec
Scale	kHz, cm/s, m/s
Automatic moving tissue suppression	Yes
Max. Color Doppler Frame Rate	> 450 frames/sec

### Power Doppler Mode (PD)

Screen Formats	2D+PD (Single, Dual, Quad)
Display Modes:	
<ul style="list-style-type: none"> <li>• Simultaneous dual mode: 2D/2D+PD</li> <li>• Triplex mode: 2D+PD/PW</li> <li>• Volume Mode: 3D+PD</li> </ul>	
PD coding	256 color steps
PD window size	Lateral: maximum to minimum B mode scan angle Axial: B-scan range
Display mode	P (power)
Wall motion Filter	7 steps (low1, low2, mid1, mid2, high1, high2, max)
Smoothing Filter	Rising edge: 12 steps Falling edge: 12 steps
Gain Control	+15 dB to -15 dB, 0.2 dB steps
PD Ensemble	7 to 31
PD Line Density	10 steps
Pulse repetition frequency	150 Hz to 20.5 kHz
PD Map	8 different color codes for each probe
Frequency range	1 to 18 MHz depending on the probe, adjustable in 3 steps (low, mid, high)
Flow Resolution	4 steps (low, mid1, mid2, high)
Balance	From 25 to 225 in 41 steps
Artifact suppression	Yes



## Scanning Parameters (cont.)

### HD-Flow

Screen Formats	2D+HDF (Single, Dual, Quad)
Display Modes:	<ul style="list-style-type: none"> <li>• Simultaneous dual mode: 2D/2D+HDF</li> <li>• Triplex mode: 2D+HDF/PW; 2D/M+MHDF</li> <li>• Volume mode: 3D+HDF</li> </ul>
HD-Flow Coding Steps	256 color steps
HD-Flow window size: lateral	Maximal to minimal B mode scan angle; axial: B-scan range
Display mode	P (power)
Wall Motion Filter	7 steps (low1, low2, mid1, mid2, high1, high2, max)
Smoothing Filter	12 steps rising edge 12 steps falling edge
Gain Control	+15 dB to -15 dB, 0.2 dB steps
HD-Flow Ensemble	7 to 31
HD-Flow Line Density	10 steps
Pulse repetition frequency	150 Hz to 20.5 kHz
HD-Flow Map	8 different color codes for each probe
Frequency Range	1 to 18 MHz depending on the probe adjustable in three steps (low, mid, high)
Flow Resolution	4 steps (low, mid1, mid2, high)
Balance	From 25 to 225
Artifact suppression	Yes

### Tissue Doppler Mode (TD)

Screen Formats	2D+TD (Single, Dual, Quad)
Display Modes	Simultaneous dual mode: 2D/2D+TD; Triplex mode: 2D+TD/PW, 2D/M+MTD;
TD coding steps	65536 color steps
Depth range	Axial: 0 to B-scan range Lateral: 0 to B-scan-range
Baseline shift	17 steps
Inversion of color direction	Yes
Smoothing Filter	12 steps rising time, 12 steps falling time
Gain Control	+15 dB to -15 dB, 0.2 dB steps
Line Density (color line density)	10 steps
Ensemble (Color shots per line)	3 to 31
Flow Resolution	4 steps (low, mid1, mid2, high)
Pulse repetition frequency	150 Hz to 20.5 kHz
TD Map	4 different color codes for each probe
Frequency range	1 to 18 MHz depending on the probe, adjustable in 3 steps (low, mid, high)

Balance	From 25 to 225
Max. meas. velocity	4.23 m/sec
Min. meas. velocity	0.3 cm/sec
Display Mode	V (velocity)
Scale	kHz, cm/s, m/s

### Volume Scan Module

Vol. scan size: max. 64 MB for gray volumes, max. 90 MB for color volumes; The required memory space depends on scan parameters (VOL-box size and quality (low, mid1, mid2, high1, high2, max). Typical: 0.8-5 MB

Lines/2D-image: max. 1024 (typ. 80 to 350)

2D-images/volume: Up to 4096 (Acquisition mode dependent)

VOL-Frames/sec.: max. 46 (typ. 4-8); The frame rate depends on scan parameters: VOL-box size, quality and probe

4D Volume Cine: up to 400 volumes, up to 512 MB

Display of sectional plane images: synchronous with control seeing, arbitrary movement in volume, monitored position in volume

Rotation: 360°, 1° or 3° increments (X-, Y- and Z-axis)

Magnification. Adjustable from 0.3 to a factor of 4.00

Acquisition Modes:

- 3D Static:
  - 3D (2D incl. CRI)
  - 3D/PD (incl. CRI)
  - 3D/CFM (incl. CRI)
  - 3D B-Flow
  - 3D/HD-flow incl. CRI
  - 3D Contrast† (Coded PI, CCIS)
- 4D:
  - 4D Real Time
  - 4D Biopsy
  - VCI-A
  - VCI-OmniView
  - STIC

- STIC:
  - Fetal Cardio
  - STIC Angio: B/Power Doppler (incl. CRI)
  - STIC CFM: B/Color Doppler (incl. CRI)
  - STIC HD-Flow: B/HD-Flow (incl. CRI)
  - STIC B-Flow
  - STIC TD
  - STIC with Contrast†

Visualization Modes:

- Render
  - 3D/4D Rendering (diverse surface and intensity projection modes)
  - SonoRender*live*
- Sectional Planes
  - Multiplanar
  - OmniView, actual and projected view
  - Niche
  - SonoVCAD*labor*
- TUI (Tomographic Ultrasound Imaging) (overview image+parallel slices)
  - TUI Standard
  - SonoVCAD*heart*

## Scanning Parameters (cont.)

### Volume Scan Module (cont.)

Visualization Modes:	
<ul style="list-style-type: none"> <li>Volume Analysis           <ul style="list-style-type: none"> <li>- VOCAL: semi-auto/ manual segmentation tool (segmentation using touch screen), (3D Static only) + Threshold Volume: measure volume below and above a threshold</li> <li>- SonoAVC<i>follicle</i> (Sono Automated Volume Count)</li> <li>- SonoAVC<i>antral</i></li> <li>- SonoAVC<i>generic</i></li> </ul> </li> <li>VCI (Volume Contrast Imaging)</li> <li>H<i>Dlive</i>: 1 free moveable Light sources</li> </ul>	
Render Modes:	
<ul style="list-style-type: none"> <li>H<i>Dlive</i></li> <li>Color</li> <li>Mix Mode of two render modes</li> <li>Surface Texture</li> <li>Surface Smooth</li> <li>Surface Enhanced</li> </ul>	<ul style="list-style-type: none"> <li>Transparency modes: max- min- and X-ray</li> <li>Gradient Light</li> <li>Inversion</li> <li>Glass Body</li> <li>Light</li> </ul>
Display graphics:	
<ul style="list-style-type: none"> <li>Rotation axis, center point</li> <li>ROI box, 3D Frame</li> <li>Temporary display of onscreen controls (rotation, translation)</li> </ul>	
Gray maps: Slices: 21 (18 basic curves and 3 User-defined (pre, post) 3D Image: one general map adjustable with bright (-50 to +50) & contrast (-50 to +50))	
Tint maps: Slices: 10; 3D image: 10	
Depth render maps: 3	

### BF (B-Flow)

Screen Formats	Single (BF), Dual (BF+BF), Quad (BF+BF+BF+BF)
Display Modes	BF, Update: BF/PW
Acc. Power range	1 – 100
Scan angle	Taken from 2D
Gain range	+15 to -15 dB
Gray scale values	8 bit
SRI	Taken from 2D
Persistence filter	8 steps (pre)
S./PRI	1.00, 1.50, 2.00, 3.00, 4.00, 5.00
Quality	3 steps (pre) low, norm, high
Enhance	6 steps (pre) 0, 1, 2, 3, 4, 5
Gray maps	21 (18 basic maps and 3 User-defined maps)
Tint maps	10
Dynamic	12 different dynamic curves C1 – C12
Accumulation	Off, 0.20, 0.35, 0.50, 0.75, 1.00, 1.50, Infinite
Background	0, 1, 2

### Contrast Imaging†

Acc. Power range	1 – 100
Scan angle	Taken from 2D
Gain range	+15 to -15 dB
Gray scale values	32 bit
SRI	Taken from 2D
Persistence filter	8 steps (pre)
S./PRI	1.00, 1.50, 2.00, 3.00, 4.00, 5.00
Quality	3 steps (pre) low, norm, high
Enhance	6 steps (pre) 0, 1, 2, 3, 4, 5
Gray maps	21 (18 basic maps and 3 User-defined maps)
Tint maps	10
Dynamic	12 different dynamic curves C1 – C12
Accumulation	Off, 0.20, 0.35, 0.50, 0.75, 1.00, 1.50, Infinite
Background	0, 1, 2
Time Delay	0, 0.5, 1, 2, 3, .....10
Display Modes:	
<ul style="list-style-type: none"> <li>Coded PI</li> <li>Coded PI: CIS</li> </ul>	<ul style="list-style-type: none"> <li>Coded PI: CCIS</li> </ul>
Screen Formats:	
<ul style="list-style-type: none"> <li>Code PI: Single (B), Dual (B+B), Quad (B+B+B+B)</li> <li>CIS: Dual simultan (2D+Coded PI)</li> <li>CCIS: Single (B), Dual (B+B), Quad (B+B+B+B)</li> </ul>	

### Elastography

Probes:	
<ul style="list-style-type: none"> <li>RIC5-9D</li> <li>11L-D</li> </ul>	<ul style="list-style-type: none"> <li>IC5-9-D</li> <li>ML6-15D</li> </ul>
Acoustic Power range: 1 – 100	
Tx Frequency: 3 (penet/norm/resol)	
Transparency: 51 steps (0,5, 10, ...255)	
Soft Compress:	
<ul style="list-style-type: none"> <li>Range: 0-9</li> </ul>	<ul style="list-style-type: none"> <li>Step Size: 1</li> </ul>
Hard Compress	
<ul style="list-style-type: none"> <li>Range: 0-9</li> </ul>	<ul style="list-style-type: none"> <li>Step Size: 1</li> </ul>
PRF: 10, 15, 25, 40, 60, 85 Hz	
Elasto Maps: 8	
Persistence:	
<ul style="list-style-type: none"> <li>Range: 1-9</li> </ul>	<ul style="list-style-type: none"> <li>Step Size: 1</li> </ul>
Line Dens.: Range: 1-2	
Filter Axial:	
<ul style="list-style-type: none"> <li>Range: 1-9</li> </ul>	<ul style="list-style-type: none"> <li>Step Size: 1</li> </ul>
Filter Lateral:	
<ul style="list-style-type: none"> <li>Range: 1-21</li> </ul>	<ul style="list-style-type: none"> <li>Step Size: 2</li> </ul>
Window Length:	
<ul style="list-style-type: none"> <li>Range: 8-25</li> </ul>	<ul style="list-style-type: none"> <li>Step Size: 1</li> </ul>
Screen Formats:	
<ul style="list-style-type: none"> <li>Single (2D/Elasto)</li> <li>Dual (2D/Elasto+2D/Elasto)</li> <li>Quad (2D/Elasto+2D/Elasto+2D/Elasto+2D/Elasto)</li> </ul>	
Elastography Analysis	
Elastography Ratio Measurement	

## Scanning Features

### Coded Excitation (CE)

Available on the following probes:

- RIC5-9-D
- 11L-D
- C4-8-D
- ML6-15-D
- RAB6-D

### Coded Harmonic Imaging

Available on all probes

### Focus Frequency Composite (FFC)

Available on the following probes:

- IC5-9-D
- RIC5-9-D
- C1-5-D
- C4-8-D
- RAB6-D
- 9L-D

### Compound Resolution Imaging (CRI)

Available on all probes, except:

- 3Sp-D
- S4-10-D

### Speckle Reduction Imaging (SRI II)

Available on all probes

### Virtual Convex

- S4-10-D
- RSP6-16-D
- 11L-D
- ML6-15-D
- 9L-D
- 3Sp-D

### Wide Sector

- IC5-9-D
- C4-8-D
- C1-5-D
- RIC5-9-D
- RAB6-D
- C4-8-D

## Measurements Tool

### Generic measurements

Distance:

- Distance (Point to Point)
- 2D Trace (Point Length)
- Distance (Line to Line)
- Stenosis (% Dist.)
- 2D Trace (Trace Length)
- Ratio D1/D2

Area/Circumference:

- Ellipse
- Stenosis (%Area)
- Trace (Line)
- Area (2 Dist.)
- Trace (Point)
- Ratio A1/A2

Volume: following Methods:

- 1 Distance
- 3 Distance
- 1 Ellipse
- Multiplane-Planimetric
- 1 Dist. + Ellipse
- Volume (3D only)

Angle:

- Angle (3 Point)
- Angle (2 Line)

M-Mode:

- Distance (Point to Point)
- HR
- Time
- Stenosis (% Dist.)
- Slope
- IMT
- Vessel Diam.
- Stenosis Diam.

PW Doppler Mode:

- Auto & Manual Trace:
  - PS (Peak Systole)
  - ED (End Diastole)
  - MD (Mid. Diastole)
  - PS/ED (Ratio)
  - PI (Pulsatility Index)
  - RI (Resistance Index)
- Vol. Flow
- PGmax, PGmean
- TAmx (Time avg. max. Velocity)
- TAmx (Time avg. mean Velocity)
- VTI (Velocity Time Integral)
- Heart Rate

Single Measurements:

- Velocity
- PS/ED
- Acceleration
- Time
- RI
- HR
- PI
- PS
- ED

### Abdomen calculations

Liver	Gallbladder
Pancreas	Spleen
Kidney (right/left)	Renal Artery (right/left)
Aorta (Proximal, Mid, Distal)	Portal Vein
Vessel	Bladder Volume
Summary Reports	

### Small part default calculations

Thyroid (right/left)
Testicle (right/left)
Dorsal Penile Artery (right/left)
Vessel
Summary Reports

### Small part breast calculations

Lesion 1-5 (right/left)
Summary Reports

## Measurements Tool (cont.)

### Obstetrics calculations

Fetal Biometry
Early Gestation
Fetal Long Bones
Fetal Cranium
NT Method: SonoNT/Manual
AFI
Uterus
Ovary right/left
Umbilical Vein
Placenta Volume
Ductus venosus: S, D, a, PI, PLI, PVIV
Doppler measurements: Ductus Art., Ductus Ven., Ao, Carotid, MCA, Celiac Artery, Superior Mesenteric Artery, Umbilical Art., Umbilical Vein, FHR, Uterine Art.
Gestational Age Calculation
Gestational Growth Calculation
Fractional limb Volume
Fetal Weight (FW) Estimation
Fetal Trend Graph
Multi-Gestational Calculation & Fetal Compare
Calculation and Ratios
Fetal Qualitative Description (Anatomical survey)
Fetal Environmental Description (Biophysical profile)
Summary Reports

### Obstetrics Fetal Echo

Chambers
Thorax
Aorta/LVOT
Pulmonary/RVOT
Venous
FHR
Tricuspid valve
Mitral Valve
Aortic
Pulmonary
LPA
RPA
Ductus Art.
Cardiac Output
LT TEI
RT TEI
Ductus Ven.
Umbilical Vein
Pulmonary Veins
Summary Reports

### Obstetrics Z-scores

Calculation of Z-scores for:	
• Long Axis	• Obl. Short axis
• Aortic Arch	• 4 Chambers
• Short Axis	• Summary Reports

### Cardiology calculations

2D Mode:	
• LV Simpson (Single & Bi-Plane)	
• Volume (Area Length)	
• LV-Mass (Epi & Endo Area, LV Length)	
• LV (RVD, IVS, LVD, LVPW)	
• LVOT Diameter	
• RVOT Diameter	
• MV (Dist A, Dist B, Area)	
• TV (Diameter)	
• AV/LA (Aortic Valve/Left Atrium)	
• PV (Diameter)	
M-Mode:	
• LV (IVS, LVD, LVPW, RVD)	
• AV/LA (Ao Root Diam, LA Diam, AV Cusp Sep., Ao Root Ampl)	
• MV(D-E, E-F Slope, A-C Interval, EPSS)	
• HR (Heart Rate) Atrial HR	
PW-Mode:	
• MV (Mitral Valve)	
• AV (Aortic Valve), TV (Tricuspid Valve)	
• PV (Pulmonary Valve)	
• LVOT & RVOT Doppler (Left & Right Ventricle Outflow Tract)	
• Pulmonic Veins	
• PAP (Pulmonary Artery Pressure measurement)	
• HR (Heart Rate)	
• TEI-Index	
C-Mode:	
• PISA	
Others:	
• Diast. Vol (Bi)	• Mean Gradient
• Syst. Vol. (Bi)	• Mean Gradient
• Stroke Volume	• Acceleration
• Volume Flow	• VTI
• Cardiac Output	• TVA
• Ejection Fraction	• PG
• Fractional Shortening	• PHT
• Myocardial Thickness	• MVA
• LA/Ao Ratio	• AVA
• E/A Peak	• ERO
• Peak Gradient	• CVP (Cardio Vascular
• Acceleration	• Profile) Score
Summary Reports	

### Transrectal calculations

Prostate
Vessel
Summary Reports incl. PSAD, PPSA(1), PPSA(2) calculation

## Measurements Tool (cont.)

### Vascular

Left/Right CCA (Common Carotid Artery)
Left/Right ICA (Internal Carotid Artery)
Left/Right ECA (External Carotid Artery)
Left/Right Vertebral Artery
Left/Right Subclav.
Left/Right Bulb
Vessels
Summary Reports

### Gynecology

Uterus
Left/Right Ovary
Left/Right Follicle
Fibroid
Endometrial thickness (Dist, Double Dist.)
Cervix Length
Left/Right Ovarian Artery
Left/Right Uterine Artery
Vessels
Pelvic Floor
Left/Right Ovarian Cyst
Left/Right Ovarian Mass
Left/Right Adnexal Cyst
Generic Cyst
Left/Right Adnexal Mass
Generic Mass
Bladder (Length/Width/Height/Vol)
FHR
GYN IOTA LR2 Model. May not be available in all countries
Summary Reports

### Pediatrics

Left/Right Hip Joint
Pericallosal Artery
Summary Report

### Cephalic

Left/Right ACA (Anterior Cerebral Artery)
Left/Right MCA (Middle Cerebral Artery)
Left/Right PCA (Posterior Cerebral Artery)
Basilar Artery
A-Com. A (Anterior Com. Artery)
P-Com. A (Posterior Com. Artery)
Left/Right CCA (Common Carotid Artery)
Left/Right ICA (Internal Carotid Artery)
Left/Right Vertebral Artery
Vessels
Summary Reports

### OB Tables

Age Tables:

- AC: ASUM, CFEF, Hadlock\_82, Hadlock\_84, Hansmann, Hobbins, Jeanty, JSUM, Kurmanavicius, Merz, Nicolaides, Shinozuka, Siriraj, Tokyo
- AD: Persson
- APAD: Merz
- APTD: Hansmann
- APTDxTTD: Shinozuka, Tokyo

- BOD: Jeanty
- BPD: ASUM, ASUM (old), Campbell, CFEF, Chitty (outer-outer) (outer-inner), Eik-Nes, Hadlock\_82, Hadlock\_84, Hansmann, Hobbins, Jeanty, Johnsen, JSUM, Kurmanavicius, Kurtz, Leung, McLennanPersson, Merz, Nicolaides, OSAKA, Rempen, Sabbagha, Shinozuka, Siriraj, Tokyo, Verburg
- CEREB: Chitty, Goldstein, HILL, Hobbins, Nicolaides, Verburg
- CLAV: YARKONI
- CRL: ASUM, DAYA, Eik-Nes, Hadlock, Hansmann, JSUM, McLennan, Persson, Pexters, Nelson, OSAKA, Rempen, Robinson, Robinson\_BMUS, Sahota, Shinozuka, Tokyo, Verburg
- Hadlock\_84, Hansmann, Hobbins, Hohler, Jeanty, JSUM, Kurmanavicius, Leung, Persson, Merz, Nicolaides, O'Brien, OSAKA, Shinozuka, Siriraj, Tokyo, WARDA, Johnsen
- FTA: OSAKA
- FIB: Jeanty
- GS: Hansmann, Hellman, Holländer, Rempen, Tokyo
- HC: ASUM, CFEF, Chitty, Hadlock\_82, Hadlock\_84, Hansmann, Jeanty, Kurmanavicius, Leung, Merz, Nicolaides, Siriraj, Johnsen
- HL: ASUM, Hobbins, Jeanty, Merz, OSAKA
- LV: Tokyo
- MAD: Eik-Nes, eSnurra, Kurmanavicius
- OFD: ASUM, Chitty, Hansmann, Jeanty, Kurmanavicius, Merz, Nicolaides
- RAD: Jeanty, Merz
- TIB: Jeanty Merz
- TAD: CFEF, Merz
- TTD: Hansmann
- ULNA: Jeanty, Merz

Growth Tables:

- AC: ASUM, CFEF, Chitty, Hadlock, Hansmann, Jacot-Guillarmod, Jeanty, JSUM, Lai\_Yeo, Kurmanavicius, Lessoway, Leung, Merz, Nicolaides, Shinozuka, Siriraj, Tokyo, Verburg, Johnsen, Medvedev, Stork, Intergrowth
- AD: Persson
- AFI: Moore
- Aorta: Vmax: Rizzo
- APAD: Merz
- APTD: Hansmann
- APTDxTTD: Shinozuka\_SD
- AxT: Shinozuka, Tokyo
- BOD: Jeanty
- BPD: ASUM, Campbell, CFEF, Chitty, Eik-Nes, Hadlock, Hansmann, Jacot-Guillarmod, Jeanty, JSUM, Kurmanavicius, Lai\_Yeo, Lessoway, Leung, Persson, McLenna, Merz, Nicolaides, OSAKA, Sabbagha, Shinozuka, Siriraj, Tokyo, Verburg, Medvedev, Stork, Intergrowth
- CLAV: YARKONI
- CM: Nicolaides
- CRL: ASUM, Hadlock, Hansmann, JSUM, McLennan, Persson, OSAKA, Robinson, Robinson 1993, Shinozuka, Tokyo, Pexters, Medvevev

## Measurements Tool (cont.)

### OB Growth Tables (cont.)

---

- DV a/S: JSUM
  - DV PI: Baschat, JSUM
  - DV PLI: Baschat
  - DV PVIV: Baschat
  - DV S/a: Baschat
  - FL: ASUM, CFEF, Chitty, Eik-Nes, Hadlock, Hansmann, Jacot-Guillarmod, Jeanty, JSUM, Kurmanavicius, Lessoway, Lai\_Yeo, Lessoway, Leung, Persson, Merz, Nicolaides, O'Brien, OSAKA, Shinozuka, Siriraj, Tokyo, Verburg, WARDA, Johnsen, Medvedev, Stork, Intergrowth
  - FTA: OSAKA
  - FIB: Chitty, Jeanty, JFFSD, Siriraj
  - FWg: Alexander
  - Foot: Chitty
  - GS: Hellman, Nyberg, Rempen, Tokyo
  - HC: ASUM, CFEF, Chervernak, Chitty, Hadlock, Hansmann, Jacot-Guillarmod, Jeanty, Kurmanavicius, Lai\_Yeo, Lessoway, Leung, Merz, Nicolaides, Siriraj, Verburg, Johnsen, Medvedev, Stork, Intergrowth
  - HL: ASUM, Chitty, Jeanty, Lai\_Yeo, Merz, JFFSD, OSAKA, Siriraj, Medvedev
  - IVC PLI: JSUM
  - Lt.Tei(ICT,IRT), Lt.Tei(a,b): Bhorat
  - Lung Area Left/Right: Peralta
  - LV: Tokyo
  - MCA CP: Ebbing
  - MCA PI: Ebbing
  - MCA PI, RI: JSUM, Bahlman
  - MCA PV: Mari
  - MAD: Eik-Nes, eSnurra, Kurmanavicius
  - MV E/A: HARADA
  - NBL: BUNDUKI, SONEK, Medvedev
  - OFD: ASUM, Chitty, Hansmann, Jeanty, Kurmanavicius, Merz, Nicolaides, Medvedev, Intergrowth
  - MainPA Vmax: Rizzo
  - RAD: Chitty, Jeanty, JFFSD, Merz, Siriraj
  - SAG. AP: Malinger
  - SAG. CC: Malinger
  - TAD: CFEF, Jacot-Guillarmod, Merz
  - TC: Chitkara
  - TCD: Goldstein, Hill, Jacot-Guillarmod, Nicolaides, Verburg
  - TIB: Chitty, Jeanty, JFFSD, Merz, Siriraj
  - TTD: Hansmann
  - TV E/A: HARADA
  - ULNA: Chitty, Jeanty, JFFSD, Merz, Siriraj
  - UmbArt PI: Ebbing, JSUM, Merz
  - UmbArt RI: JSUM, Merz, Kurmanavicius
  - UtArtPI: Gomez, Merz
  - UtArtRI: Merz
  - Vermis A: Malinger
  - Vermis C: Malinger
  - Fractional Limb Avol/Tvol: Lee
- 

### Fetal weight Estimation (EFW)

---

- Campbell (AC)
- Hadlock (AC, BPD)
- Hadlock 1 (AC, FL)
- Hadlock 2 (BPD, AC, FL)
- Hadlock 3 (HC, AC, FL)
- Hadlock 4 (BPD, HC, AC, FL)
- Hansmann (BPD, TTD)
- Lee (AVOL; AC, AVOL; AC, BDP, AVOL; TVOL; AC, TVOL; AC, BDP, TVOL)
- Merz (AC, BPD)
- Osaka (BPD, FTA, FL)
- Persson (BPD, MAD, FL)
- Persson 2, Schild (HC, AC, FL)
- Shepard (AC, BPD)
- Shinozuka 1 (BPD, ADTP, TTD, FL)
- Shinozuka 2 (BPD, FL, AC)
- Shinozuka 3 (BPD, APTD, TTD, LV)
- Tokyo (BPD, APTD, TTD, FL)

### Gestational Age by EFW

- Hadlock, JSUM 2001, Osaka, Shinozuka, Tokyo
- 

### Fetal Weight Growth FWG

- Alexander, Ananth, Bourgoigne, Brenner, CFEF, Doubilet, Ego, Eik-Nes, Hadlock, Hansmann, Hansmann (86), Hobbins/Persutte, Johnsen, Jsum 2001, Kramer, Persson, Osaka, Shinozuka, Tokyo, Williams, Yarkoni
- 

### Fetal ratios

---

CI (BPD/OFD) (Hadlock)

---

FL/AC (Hadlock)

---

FL/BPD (Hohler)

---

FL/HC (Hadlock)

---

HC/AC (Campbell)

---

Va/Hem (Nicolaides)

---

Va/Hem (Hansmann)

---

Vp/Hem (Nicolaides)

---

LHR (Peralta)

---

CVR (Peranteau)

---

## Probes

### C1-5-D

Wide Band Convex Probe	
Applications	Abdomen, OB, GYN
Maximum Bandwidth (-20dB)	2-5 MHz
Number of Elements	192
Convex Radius	56.1 mm
FOV	113°
Foot Print	69.3 x 17.2 mm
Depth	Max. 30 cm
Center Frequency	3.4 MHz
B-Mode Frequency	2.78 - 3.70 MHz
Doppler Frequency	2.00 - 3.23 MHz
Harmonic Frequency	2.78 - 3.70 MHz
Biopsy Guide Available	Multi-Angle, disposable with reusable bracket

### C4-8-D

Wide Band Convex Probe	
Applications	Abdomen, OB, GYN, Pediatrics
Maximum Bandwidth (-20dB)	2-8 MHz
Number of Elements	192
Convex Radius	39.1 mm
FOV	95°
Foot Print	55.2 x 17.6 mm
Depth	Max. 26 cm
Center Frequency	4.4 MHz
B-Mode Frequency	3.23 - 5.88 MHz
Doppler Frequency	3.03 - 4.00 MHz
Harmonic Frequency	2.56 - 3.03 MHz
Biopsy Guide Available	Multi-Angle, disposable with reusable bracket

### IC5-9-D

Wide Band Convex Probe	
Applications	OB, GYN, Transrectal
Maximum Bandwidth (-20dB)	4-9 MHz
Number of Elements	192
Convex Radius	10.8 mm
FOV	179°
Foot Print	21.2 x 17.2 mm
Depth	Max. 16 cm
Center Frequency	5.8 MHz
B-Mode Frequency	4.55 - 7.69 MHz
Doppler Frequency	5.00 - 6.25 MHz
Harmonic Frequency	3.45 - 3.85 MHz
Biopsy Guide Available	Single-Angle, Reusable and disposable

### 11L-D

Wide Band Linear Probe	
Applications	Small Parts, Pediatrics, MSK, Peripheral Vascular, Breast
Maximum Bandwidth (-20dB)	4-10 MHz
Number of Elements	192
FOV	37.4 mm
Foot Print	46.9 x 14.4 mm
Depth	Max. 11 cm
Center Frequency	7.3 MHz
B-Mode Steering Angle	7°/14°/20°
Doppler Steering Angle	7°/14°/20°
B-Mode Frequency	6.67 - 10.00 MHz
Doppler Frequency	5.26 - 7.14 MHz
Harmonic Frequency	4.55 - 5.00 MHz
Biopsy Guide Available	Multi-Angle, disposable with reusable bracket

### 9L-D

Wide Band Linear Probe	
Applications	Small Parts, Pediatrics, MSK, Peripheral Vascular, OB
Maximum Bandwidth (-20dB)	3-8 MHz
Number of Elements	192
FOV	43 mm (width)
Foot Print	53.0 x 14.1 mm
Depth	Max. 14 cm
B-Mode Steering Angle	7°/14°/20°
Doppler Steering Angle	7°/14°/20°
Center Frequency	5.5 MHz
B-Mode Frequency	4.55 - 10.00 MHz
Doppler Frequency	3.70 - 5.26 MHz
Harmonic Frequency	2.86 - 2.86 MHz
Biopsy Guide Available	Multi-Angle, disposable with reusable bracket

### ML6-15-D

Wide Band Matrix Linear Probe	
Applications	Small Parts, Peripheral Vascular, Pediatrics, MSK, Breast
Maximum Bandwidth (-20dB)	4-13 MHz
Number of Elements	1008
FOV	49.6 mm (width)
Foot Print	60.7 x 16 mm
Depth	Max. 12 cm
B-Mode Steering Angle	7°/14°/20°
Doppler Steering Angle	7°/14°/20°
Center Frequency	9.0 MHz
B-Mode Frequency	8.33 - 11.11 MHz
Doppler Frequency	6.25 - 9.09 MHz
Harmonic Frequency	5.00 - 6.25 MHz
Biopsy Guide Available	Multi-Angle, disposable with reusable bracket

**RAB6-D**

Wide Band Convex Volume Probe	
Applications	Abdomen, OB, GYN, Pediatrics
Maximum Bandwidth (-20dB)	2-8 MHz
Number of Elements	192
Convex Radius	46.8 mm
Volume Sweep Radius	24.11 mm
FOV	90° (B), 90° x 85° (Volume scan)
Foot Print	62.2 x 34.0 mm
Depth	Max. 26 cm
Center Frequency	4.7 MHz
B-Mode Frequency	3.23 – 6.67 MHz
Doppler Frequency	3.03 – 5.00 MHz
Harmonic Frequency	2.63 – 3.33 MHz
Biopsy Guide Available	Multi-Angle, disposable with reusable bracket

**RIC5-9-D**

Wide Band Convex Volume Probe	
Applications	OB, GYN, Transrectal
Maximum Bandwidth (-20dB)	4-9 MHz
Number of Elements	192
Convex Radius	11.6 mm
Volume Sweep Radius	11.6 mm
FOV	180°(B), 180° x 120° (Volume scan)
Foot Print	22.4 x 22.6 mm
Depth	Max. 16 cm
Center Frequency	6.6 MHz
B-Mode Frequency	4.55 – 7.69 MHz
Doppler Frequency	5.00 – 6.25 MHz
Harmonic Frequency	3.45 – 3.85 MHz
Biopsy Guide Available	PEC63, Single-Angle, Reusable, Disposable, disposable with latex cover

**RSP6-16-D**

Wide Band Linear Volume Probe	
Applications	Small Parts, Pediatrics, MSK, Peripheral Vascular, Breast
Maximum Bandwidth (-20dB)	6-18 MHz
Number of Elements	192
Volume Sweep Radius	80.7 mm
FOV	37.4 mm (B), 37.4 mm x 29° (Volume scan)
Foot Print	48.6 x 55.9 mm
Depth	Max. 8 cm
B-Mode Steering Angle	7°/14°/20°
Doppler Steering Angle	7°/14°/20°
Center Frequency	11.5 MHz
B-Mode Frequency	8.33 – 12.50 MHz
Doppler Frequency	6.25 – 8.33 MHz
Harmonic Frequency	5.26 – 5.26 MHz
Biopsy Guide Available	PEC75, Single-Angle, Reusable and Disposable

**3Sp-D**

Wide Band Phased Array Probe	
Applications	Abdominal, Cardiology, OB, Pediatrics, Cephalic
Maximum Bandwidth (-20dB)	1-5 MHz
Number of Elements	64
FOV	90°
Foot Print	23.4 x 20.2 mm
Depth	Max. 24 cm
Center Frequency	3.2 MHz
B-Mode Frequency	2.44 – 3.33 MHz
Doppler Frequency	1.85 – 2.50 MHz
Harmonic Frequency	1.92 – 1.96 MHz
Biopsy Guide Available	Multi-Angle, disposable with reusable bracket

**S4-10-D**

Wide Band Phased Array Probe	
Applications	Small Parts, Cardiology, Pediatrics
Maximum Bandwidth (-20dB)	4-9 MHz
Number of Elements	128
FOV	90°
Foot Print	20.0 x 15.0 mm
Depth	Max. 14 cm
Center Frequency	6.3 MHz
B-Mode Frequency	5.18 – 7.14 MHz
Doppler Frequency	3.85 – 6.25 MHz
Harmonic Frequency	4.00 – 4.35 MHz
Biopsy Guide Available	Not available



## External Inputs and Outputs

---

Connectivity on rear panel (direct access)

- VGA out
  - Network (RJ45)
  - Wireless Network interface (USB) (Option)
  - USB 2.0 (1x)
  - USB 3.0 (5x)
  - S-Video Out 1
  - HDMI
- 

Connectivity behind rear panel (access after opening):

- DVI-D out
  - S-Video Out 1
  - Audio Out
    - Left/right
  - USB 2.0 (3x)
  - RS 232: via USB to RS 232 converter
  - Ext. Device/Remote Connections:
    - Remote BW Printer via USB
    - Remote Color Printer/DVR via USB
    - Remote VCR (RS232)/DVR via USB
    - Footswitch via USB
    - ECG
-

## Safety Conformance

### The Voluson E6 is:

---

NRTL certified according IEC 60601-1 (TÜVPS)

---

Certified to CSA 22.2, 60601.1 by an SCC accredited Test Lab

---

CB-Test Report by National Certification Body

---

CE Marked to Council Directive 93/42/EEC on Medical Devices

---

Conforms to the following standards for safety:

---

IEC 60601-1 Electrical Medical Equipment

---

IEC 60601-1-2 Electromagnetic compatibility

---

IEC 62304 Software Life Cycle Processes

---

IEC 62366 Application of usability engineering to medical devices

---

EN 60601-2-37 Particular requirements for the safety of ultrasound medical diagnostic and monitoring equipment

---

IEC 601157 Declaration of acoustic output

---

ISO 10993 Biological evaluation of medical devices

---

IEC 62359 Ultrasonic - Field characterization - Test methods for the determination of thermal and mechanical indices related to medical diagnostic ultrasonic fields

---

WEEE (Waste Electrical and Electronic Equipment)

---

ROHS according to 2011/65/EU

---

†Not for sale in the USA. Not approved or cleared by the U.S. FDA.

Please contact your GE Sales Representative for information about availability in your area.

GE Healthcare  
9900 Innovation Drive  
Wauwatosa, WI 53226  
U.S.A.

**Europe**  
**GE Healthcare**  
Beethovenstr. 239  
D - 42655 Solingen  
T +49 212 2802 0  
F +49 212 2802 28

[www.gehealthcare.com](http://www.gehealthcare.com)

**APAC**  
**GE Healthcare Asia Pacific**  
4-7-127, Asahigaoka,  
Hino-shi, Tokyo  
191-8503 Japan  
T +81 42 585 5111

© 2017 General Electric Company – All rights reserved.

General Electric Company reserves the right to make changes in specifications and features shown herein, or discontinue the product described at any time without notice or obligation. Contact your GE Representative for the most current information.

\* GE, GE monogram, Voluson, HD*live*, SonoAVC, SonoVCAD, SonoNT, B-Flow, HD-Flow, and CrossXBeam<sup>CR</sup> are trademarks of General Electric Company or one of its subsidiaries.

® DICOM is the registered trademark of the National Electrical Manufacturers Association for its standards publications relating to digital communications of medical information.

GE Medical Systems Ultrasound & Primary Care Diagnostics, LLC, a General Electric company, doing business as GE Healthcare.



DOC1877666